


Polishing Cloth for ASSI® Noble™ Bipolar Forceps

Ultra Soft Polishing Cloth Two – Cloth System in 1 Cloth

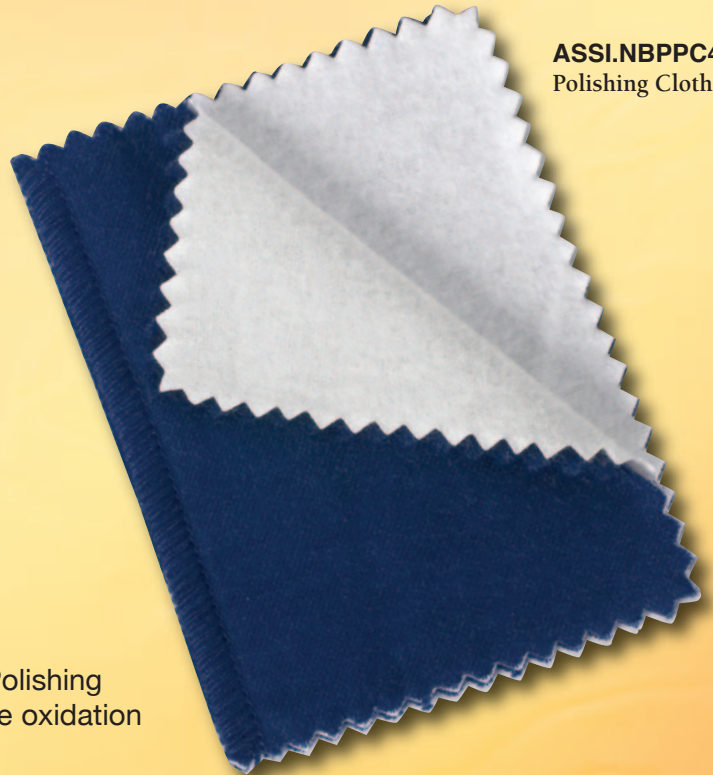

ASSI.NBPPC46
Polishing Cloth

- The ASSI® Noble™ Bipolar Forceps Polishing Cloth System may be used to remove oxidation from the Noble™ Metal Tip.
- After multiple reuse and sterilization cycles, a layer of oxidation may form on the surface of the Noble™ Bipolar Forceps Tips. Oxidation on the tip is harmless and does not interfere with the non-stick performance.
- Once you have decontaminated and cleaned the forceps use the Inner White Cloth to remove the oxidation.
- Use the Outer Blue Cloth to gently polish the tips before you autoclave.

Repeat this process when and if oxidation returns.

ASSI.NBPPC46 Noble Bipolar Forceps Polishing Cloth:
3 each 4 x 6" cloths per package



ACCURATE SURGICAL & SCIENTIFIC INSTRUMENTS®
For diamond perfect performance®

accurate surgical & scientific instruments corporation

300 Shames Drive, Westbury, NY 11590

800.645.3569 516.333.2570 fax: 516.997.4948 west coast: 800.255.9378

Info: assi@accuratesurgical.com • Orders: orders@accuratesurgical.com

www.accuratesurgical.com

Not all ASSI products shown in our literature or on our website are available for sale in Canada