

TEST-SIZE® ORCHIDOMETER

FOR QUICK AND EASY MEASUREMENT OF TESTES

For years, the medical profession has understood the correlation between testes size and function. In pediatric and adolescent endocrinology, determining testicular growth is critical for assessing disturbances of puberty, abnormal testicular development and the therapeutic effect of hormones vis-a-vis sperm production. In adults, testicular size is one of the important criteria in the accurate evaluation of male reproductive function.

Based on the Prader model, the orchidometer allows the physician to make rapid evaluations using the comparative palpation method. It offers many advantages over calipers, which can be difficult to use,


even with proper positioning of the testes. Moreover, calipers are ineffective as a diagnostic tool in many cases of undescended testes. The Test-Size® Orchidometer affords more patient comfort and is easier to use... without compromising accuracy.

Benefits of the Orchidometer:

- Facilitates an accurate diagnosis
- Easy, safe to use
- Eliminates calipers
- Minimizes patient embarrassment and discomfort
- On-the-spot evaluations
- Used in extensive clinical trials

How to use the orchidometer:

Patients should be examined in a supine position to ensure an accurate measurement. The testis is lifted between the thumb and index finger, with the skin gently stretched over the testis. Move the testis with a slight pressure of the thumb and index finger of the other hand (fig. 1) to facilitate size assessment under the scrotal skin. The testis is then compared to its corresponding orchidometer model (fig. 2). If testicular size falls between two of the model sizes, the larger should be used.

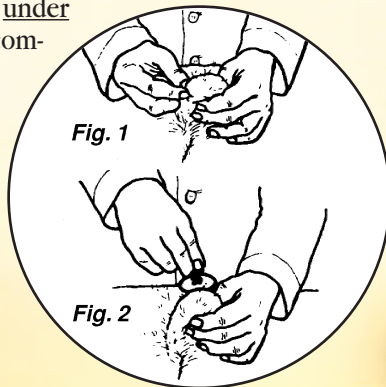

Test-Size® is a registered trademark of Remcat Trade A.B.

Available in two set sizes:

RA125

Consists of 12 models,
1 to 25ml

RA125PLUS

Consists of 14 models,
1 to 35ml.

ASSI® is the exclusive North American distributor of the Test-Size® orchidometer.

assi® 

ACCURATE SURGICAL & SCIENTIFIC INSTRUMENTS®

For diamond perfect performance®

accurate surgical & scientific instruments corporation

300 Shames Drive, Westbury, NY 11590

800.645.3569 516.333.2570 fax: 516.997.4948 west coast: 800.255.9378

Info: assi@accuratesurgical.com • Orders: orders@accuratesurgical.com • www.accuratesurgical.com

Note: Not all ASSI products shown in our literature or on our website are available for sale in Canada.